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Lækningatæki

PreCold® Mouth Spray INSTRUCTIONS FOR USE

DEVICE DESCRIPTION

PreCold® Mouth Spray targets the mouth and throat, where cold viruses take hold and multiply. PreCold® Mouth Spray works by coating the lining of the mucous membrane, creating a protective barrier. The barrier acts osmotically on the cold viruses, trapping them and preventing them from binding with human cells, helping the body to remove them naturally.

INDICATION

To be used when exposed to cold viruses or early on when cold symptoms occur. PreCold® Mouth Spray is suitable for adults and children over 4 years.

INTENDED USE

PreCold® Mouth Spray reduces the probability of catching a cold. PreCold® Mouth Spray can help shorten the duration of a cold if used at an early stage of the infection.

PRECAUTIONS

- Do not use PreCold® Mouth Spray if you are hypersensitive/allergic to any of the ingredients.
- Do not inhale when applying the spray since it may cause transient asthma-like symptoms, such as coughing and hoarseness.
- Consult a physician if you have a high fever or if your symptoms are more severe than those of a normal common cold; they could be due to something else.
- Do not use the product for more than 30 consecutive days, there is no clinical information about long-term use.
- Keep out of reach of children, to prevent accidents with small parts.

PREGNANCY AND BREASTFEEDING

There is no clinical information about use of PreCold® Mouth Spray during pregnancy and breastfeeding; please consult a physician before use.

The PreCold® Mouth Spray package contains:

- Mouth Spray consisting of bottle, pump, nozzle and protective cap.
- Solution containing glycerol, water, cod trypsin¹, ethanol (<1 %), calcium chloride, trometamol and menthol. It is sugar-free and contains no preservatives.
- This instructions for use.

PreCold® is available in one size²:

– PreCold® Mouth Spray 20 ml.

¹ Produced by Zymetech ehf., Iceland.

² The bottle is overfilled for technical reasons to ensure the sufficient number of doses.


DIRECTIONS FOR USE

How to use PreCold® Mouth Spray as prevention

If you do not have any symptoms of a cold but are likely to be exposed to the cold virus, for example when someone near you has a cold.

1. Open your mouth and aim the nozzle towards your throat (Figure 1).
2. Press down on the pump and spray 2 puffs (1 dose) every second hour up to 6 times daily during the time you are exposed to the cold virus.

Figure 1

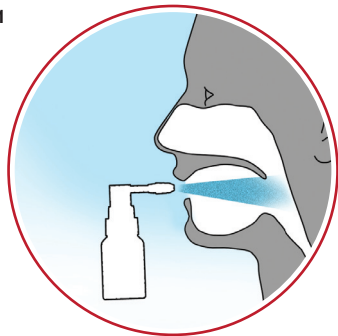How to use PreCold® Mouth Spray to help shorten the duration of a common cold

Start using PreCold® Mouth Spray as soon as possible after noticing symptoms of a cold. Continue to use it until the symptoms are relieved.

1. Open your mouth and aim the nozzle towards your throat (Figure 1).
2. Press on the pump and spray 2 puffs (1 dose) every second hour up to 6 times daily.

CARE AND STORAGE

PreCold® Mouth Spray must be used before the expiration date indicated on the product label and package.

Store PreCold® Mouth Spray at room temperature, max 25°C.

EXPLANATION OF SYMBOLS

- Consult instructions for use
- Upper limit of temperature
- Use by date
- Batch code
- Catalogue Number
- Manufacturer
- Caution, consult instructions for use

REF

CZ-MS06002001IS PreCold® Mouth Spray 20 ml

MANUFACTURER:

Enzymatica AB
IDEON Science Park
SE-223 70 Lund, Sweden
info@enzymatica.com
www.enzymatica.com

Medical Device